

AvaMeter

INSTRUCTIONS FOR USE

1

Measure the distance from the incisive papilla to the **Resting Lip**.

The dental technician will position the incisal edge of the central incisors using this measurement.


Record your reading in millimeters

2

Measure the distance from the incisive papilla to the **Smiling Lip**.

This measurement helps determine the length of the central incisors.

Record your reading in millimeters

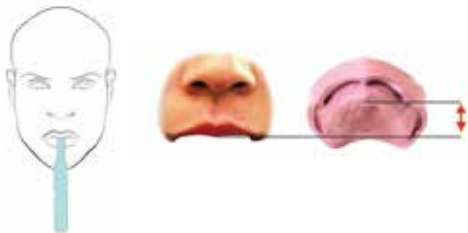

3

Measure the width of the nose from ala to ala.

This measurement helps position the canines.

Record your reading in millimeters



4

Measure the width of the eyes from pupil to pupil.

This measurement combined with a picture of the patient helps the technician select the anterior teeth.

Record your reading in millimeters



Submit all your records with your digital prescription through the AvaDent Dashboard and receive your Wagner Try-In.

For more information about the AvaDent Wagner EZ Guide Protocol, or the WTI, please review our clinical protocols on

<http://www.avadent.com/clinical/record-methods/wagner-ez-guide/>

Important Information: Follow a Cold Sterilization process for the AvaMeter based on approved ADA guidelines using OSHA approved cleaning liquids and processes. Do not use autoclave or other heat sources to sterilize the AvaMeter.